



Heated Therapy Braces Manual



Disclaimer

This product is not used to completely cure joint and nerve diseases. It is only used as an auxiliary means of related physiotherapy to relieve symptoms, promote pet activity skills, or use as pet maintenance.

PRODUCT INTRODUCTION

Petietec Targeted Heating Therapy Braces are designed to help dogs recover from arthritis, rheumatism, and postoperative procedures, providing pain relief. They are also an ideal complement to regular medication or therapy treatment.


What's it used for

- 🐾 Osteoarthritis, Hip Dysplasia, Elbow Dysplasia, Luxating patellas etc.
- 🐾 Post surgery support like TPLO, CCL and ACL surgery
- 🐾 Rehabilitation after injury
- 🐾 Decreased endurance during activity
- 🐾 Strain or sprain of the knee joint
- 🐾 Leg limping, instability, or fatigue

How dose the heat Braces works

New graphene heating film technology to produce far infrared ray heat that penetrates into bone and muscle to speed up local blood circulation, melt away chronic pain, and improves the lubricating performance of joint lubricating fluid.

- 🐾 Graphene Far-Infrared Heating
- 🐾 3 Intelligent Temperature Ranges, safe and controllable
- 🐾 USB power bank, low voltage and pet safe

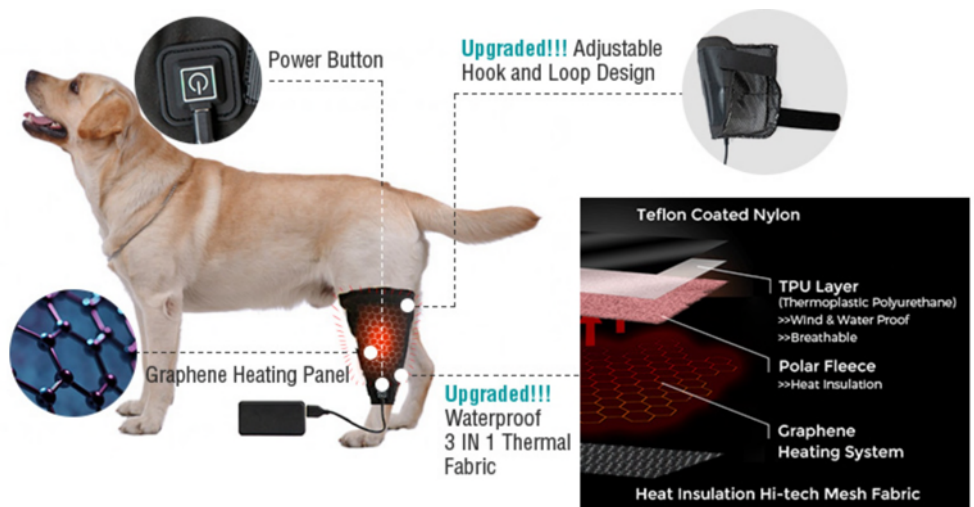

Product Feature

- 🐾 Widely approved by Professionals
- 🐾 Pets can move freely with the braces on
- 🐾 Three Heating Settings & Easy to Use
- 🐾 Targeted Heating
- 🐾 Natural, Safe and no side effect

PRODUCTION SPECIFICATIONS

Heating Therapy Braces

Product	Size	Length * Width (cm)	Number of heating panel	Material	Heating Panel Parameters	Note
Elbow/ Knee Braces	M	19*13	1	Grapnene	10*7 cm ± 1 mm 0.4-0.5A	5V 0.4-0.5A per Panel Android switch
	L	29.5*20.5	2		10*7 cm ± 1 mm*2 0.8-1.0A	
Hip Braces	M	26.5*13	1		10*7 cm ± 1 mm 0.4-0.5A	
	L	44*26	2		10*7 cm ± 1 mm*2 0.8-1.0A	
	XL	53*30	4		10*7 cm ± 1 mm*4 1.6-1.9A	
Back Brace	M	30*30	3		10*7 cm ± 1 mm*3 1.2-1.5A	
	L	35*40	4		10*7 cm ± 1 mm*4 1.6-1.9A	
	XL	55*50	4		10 *14 cm ± 1 mm*4 1.6-1.9A	
Shoulder & Neck Brace	M	45*20	4		10*7 cm ± 1 mm *4 1.6-1.9A	
	L	52*27	4		10 *14 cm ± 1 mm*4 1.6-1.9A	


Power Banks

Size	Capacity	Voltage	Note
1.7*3.4*0.6 inches/ 4.3 * 8.6* 1.5cm	3000mAh/3.7V 11.1Wh	5V	Micro USB connector, Charging time 90-120 mins

OPERATING INSTRUCTIONS

1. Therapy Vest

PACKAGE COMPONENTS

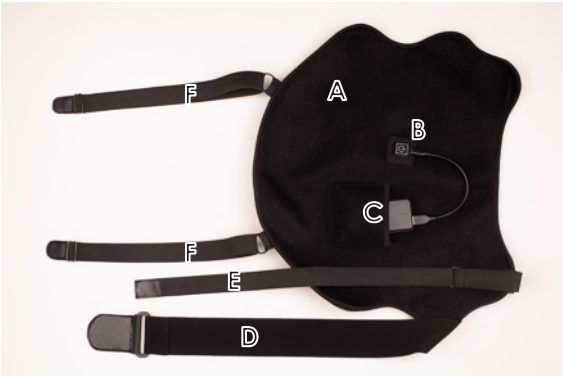

- A.Top layer made of Tpu fabric
- B.Power button
- C.Power Bank Pocket
- D.Adjustable chest band
- E.Adjustable neckband
- F.Adjustable leg band
- G.Graphene heating panel
- H.Inner layer made of Stereo Gille Fabric


HOW TO PUT ON



Place the vest on the dog's back. Attach the chest band first to keep it in place. Adjust the band to best fit.




Attach the leg band from between the rear legs to the side of the vest. Adjust the band to best fit. Repeat the other side.

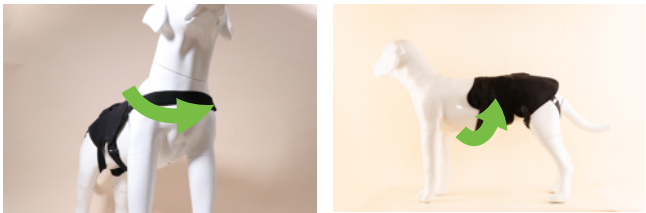

Attach the last band either around the neck or around the chest. Adjust the band to best fit.

2. Hip Heated Therapy Brace

PACKAGE COMPONENTS

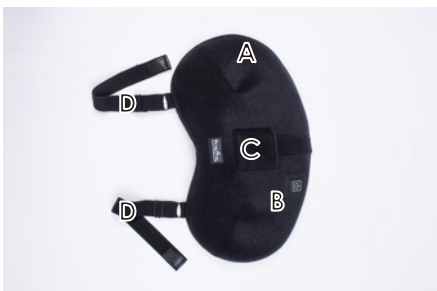

Front



Back



Collar Band

DESCRIPTION

- A.Top layer made of Tpu fabric

B.Power button

C.Power Bank Pocket

D.Adjustable Leg bands

E.Graphene heating panel
- F.Inner layer made of Stereo Gille Fabric







G.Connector

H.Adjustable Neckbands

I.Back connection band

J.Metal Hook

HOW TO PUT ON

 	Place the neckband around the dog's neck as a collar. Then position the adjustable strap along the spine.
 	Connect the hip brace to the trap. Adjust the strap to make sure the brace is at the hip position.
 	Attach the leg band from between the rear legs to the side of the vest. Adjust the band to best fit. Repeat the other side.

3. Knee/elbow Heated Therapy Brace

PACKAGE COMPONENTS



HOW TO PUT ON

		Wrap the knee brace around the dog's knee or elbow. Remember to keep the pocket on the outside. Then Connect the elastic band to both sides of knee braces.
		Connect the middle connector to the collar. Adjust the band to best fit.
		When wearing the leg brace together with the hip or back part, simply apply the velcro of the connector to the surface of hip or back brace. No need to connect to the collar one more time.
		It can also be used on front legs. Just wrap to the pain position, and connect the band.

4.Shoulder & Neck Heated Therapy Brace

PACKAGE COMPONENTS



FRONT





Back



DESCRIPTION

- A.Top layer made of Tpu fabric
- B.Power button
- C.Power Bank Pocket
- D.Elastic Neck band
- E.Elastic Left Leg band
- F.Elastic Right Leg band
- G.Graphene heating panel
- H.Inner layer made of Stereo Gille Fabric




HOW TO PUT ON

		Place the brace around the dog's shoulder and neck. Attach the neck band first to keep it in place.
		Attach the leg bands from between the front legs to the side of the brace. Adjust the band to best fit. Repeat the other side.

HEAT ON

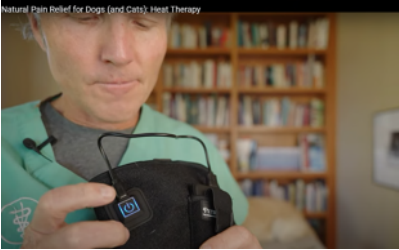


	Install the power bank or power plug with the micro USB connector. Press and hold the power button for 3 seconds to turn it on.
	Press the button to switch the temperature. The three temperature ranges: Red – (48℃ /118.4 ℉) Blue–(40℃ /104 ℉) Green–(35℃ /95 ℉)

VETERINARIAN APPROVED

		
Three Heating Settings	Targeted Heating	Pets can move with Braces on

Dr. Andrew Jones

A natural pet health advocate, author, and former practicing veterinarian who enjoys making free educational and instructional videos for pet lovers and pet parents. His natural pet health YouTube channel, Veterinary Secrets, shares short educational and instructional 'How To' videos on common pet health concerns, new discoveries and updates in conventional and holistic veterinary medicine, as well as tips, techniques and home remedies to help your dog and cat. He recommended our heated therapy braces.

		
Targeted Heating	Three Heating Settings	Comfortable Materials

Dr. Alex Avery

Veterinarian Dr. Alex Avery shares his experience and knowledge with pet owners so that we can be confident we are making the best choices for our furry family member. Dr. Alex recommends our heated braces in his videos. He thinks that the heat therapy is a natural and safe way to help dogs pain relief. And the heated therapy brace will help improve healing and treat the chronic pain of arthritis in your dog.

CUSTOMER REVIEWS



Chino the GSD had ACL Surgery

Name: Chino

Breed: German Shepherd

Age: 6

Health Issue: ACL replacement
& Arthritis

Product Used: Knee Brace,
hip brace

**"She isn't as stiff when getting
up after using them which is
amazing!"**



Tonka for Luxating Patellas

Name: Tonka

Breed: American Bully

Age: 5

Health Issue: Luxating patellas

Product Used: Back brace,
Knee brace

**"It's super easy to stay in place.
Tonk feels soooooooo good! "**



**Nella the GSD with Arthritis
in Knees**

Name: Nella

Breed: German Shepherd

Age: 9

Health Issue: Arthritis

Product Used: Knee brace,
hip brace

**"It helped my knee feel better
and I've been able to enjoy
my walks"**



Chino the GSD had ACL Surgery

Name: Sallie


Breed: Labrador

Age: 7

Health Issue: CCL/hip dysplasia

Product Used: Hip Brace

**"She has been enjoying using
it and I really think it's
helping her!"**

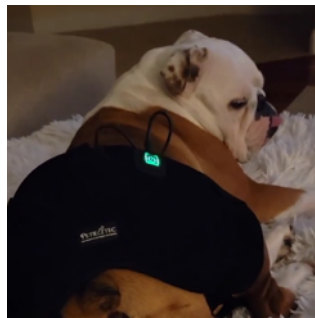

Tonka for Luxating Patellas

Name: Risotto

Breed: Bulldog

Age: 3

Health Issue: Hip dysplasia

Product Used: Hip Brace

**"Heat therapy has been so
beneficial for my hips!"**



**Nella the GSD with Arthritis
in Knees**

Name: Laika

Breed: German Shepherd

Age: 4

Health Issue: Cervical
Myelopathy

Product Used: Knee brace,
hip brace, back vest

**"It's easy to use, it's comfortable
for the dogs. They all fell
asleep. "**

HELPFUL TIPS

- 🐕 Your dog may need a few minutes to adjust to wearing the braces. Check the fit to ensure there are no areas of discomfort. Monitor the dog during the treatment period.
- 🐕 Vet suggests doing the treatment 10-20 minutes each time, 2 times a day.
- 🐕 Waterproof fabric, hand washes only.

WARNING

- 🐕 DO NOT leave the dog unattended while wearing the device.
- 🐕 DO NOT allow your dog to chew on the braces.
- 🐕 DO NOT attach to wounded skin directly.
- 🐕 DO NOT place in the microwave oven or high-pressure gas, in order to avoid fire.
- 🐕 Please stop use immediately when the device is abnormally hot to avoid damage or fire.



<https://partner.petietec.com/>
Email:petietecvendorservice@gmail.com