

## MASSAGE CONTROLLER



Attach the massage controller to the massage braces.  
Massage Controller includes an install battery.  
Connect the usb charger to the massage controller to charge.

**⏻** : Press and hold the power button for 3 seconds to turn it on. Also press the power button to turn it off.

**M** : Press the Button M to change the vibration mode. We offer 3 mode, you can see the change like "1", "2", "3" ("1" means mode 1) on the digital screen.

**+** : Press the Button + to increase the vibration intensity. We offer 3 mode, press once, the intensity increase once. When you hear the sound like "didi", it means that now is the highest vibration intensity.

**-** : Press the Button - to reduce the vibration intensity. We offer 3 mode, press once, the intensity reduce once. When you hear the sound like "didi", it means that now is the the lowest vibration intensity.

## HELPFUL TIPS

1. Your dog may need a few minutes to adjust to wearing the braces. Check the fit to ensure there are no areas of discomfort. Monitor the dog during the treatment period.
2. Vet suggests doing the treatment 10-20 minutes each time, 2 times a day.
3. Waterproof fabric, hand washes only.

## WARNING

1. DO NOT leave the dog unattended while wearing the device.
2. DO NOT allow your dog to chew on the braces.
3. DO NOT attach to wounded skin directly.
4. DO NOT place in the microwave oven or high-pressure gas, in order to avoid fire.
5. Please stop use immediately when the device is abnormally hot to avoid damage or fire.



Scan QR code for manual video

Please share to other fur babies who need these.

They deserve better lives.

✉ Email: [service@petietec.com](mailto:service@petietec.com)



## PRODUCT INTRODUCTION

Thanks for using Petietec Massage Therapy Braces. Massage may reduce stress and anxiety, increase circulation, decrease pain, and strengthen the bond between you and your dog. Also, it can help to identify new growths at an early stage.

### WHAT OUR MASSAGE THERAPY BRACES CAN DO

**Anxiety:** The hands-on contact of massage may be useful in reducing stress during a thunderstorm, fireworks show, or any situation that makes your pet nervous and restless.

**Arthritis:** The act of gentle rubbing and kneading can increase circulation to sore muscles and joints. The gentle pressure of your hands causes tissues to contract and increases local blood flow.

**Pre-Activity:** Just like human athletes, dogs, can benefit from a pre-workout warm-up. A gentle massage stimulates circulation; increases blood supply to joints, muscles, and nerves; and can even help prevent injury and post-workout soreness.

**Post-Activity:** Age, overexertion, too much inactivity, and previous injuries can all lead to postexercise stiffness and pain. Massage can help alleviate some of this discomfort.


### WHO CAN BENEFIT FROM OUR HEATING THERAPY BRACES

- ✓ Dogs with chronic pain, like arthritis and hip/elbow dysplasia
- ✓ Dogs with Muscle Stiffness/Soreness
- ✓ Dogs with a lot of exercise and training, like working dogs and agility dogs
- ✓ Senior Dogs

## OPERATING INSTRUCTIONS

### 1. BACK MASSAGE BRACE

#### PACKAGE COMPONENTS

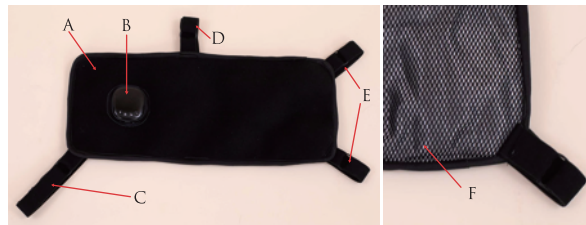

#### DESCRIPTION

- A: Top layer made of Tpu fabric
- B: Massage controller
- C: Adjustable neck band
- D: Adjustable back band
- E: Adjustable hip band
- F: Inner layer of Stereo Gille Fabric

1


### HOW TO PUT ON



1. Place the brace on the dog's leg. Attach the leg band first to keep it in place. Adjust the band to best fit
2. Attach the back band to keep it in place. Adjust the band to best fit
3. Attach the hip band from between the rear legs to the side of the brace. Adjust the band to best fit.

### 2. 2-IN-1 MASSAGE BRACE USED FOR NECK&BACK OR BACK&HIP

#### PACKAGE COMPONENTS

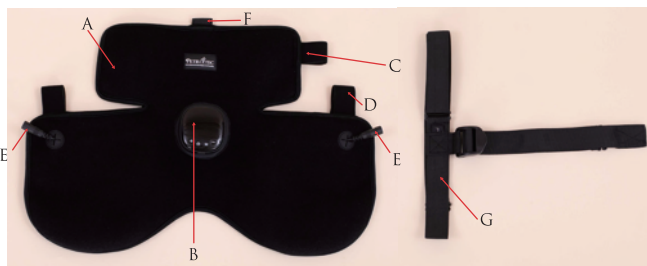

#### DESCRIPTION

- A: Top layer made of Tpu fabric
- B: Massage controller
- C: Adjustable neck/back band
- D: Adjustable leg band
- E: Used to connect Knee Massage Brace
- F: Used to connect dog collar
- G: Dog collar

### HOW TO PUT ON - USED FOR NECK & BACK



1. Place the brace on the dog's neck. Attach the neck band first to keep it in place. Adjust the band to best fit
2. Attach the leg band from between the front legs to the side of the brace. Adjust the band to best fit.

2


### HOW TO PUT ON -USED FOR BACK & HIP



1. Place the brace on the dog's back and hip. Attach the back band first to keep it in place. Adjust the band to best fit
2. Attach the leg band from between the rear legs to the side of the brace. Adjust the band to best fit.
3. Put the collar on your dog. Attach the collar to the band used to connect dog collar. Adjust the length of the collar to best fit.

### 3. ACCESSORIES: KNEE MASSAGE BRACE

#### PACKAGE COMPONENTS

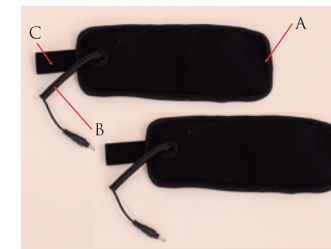

#### DESCRIPTION

- A: Top layer made of Tpu fabric
- B: Used to connect 2-in-1 Massage Brace used for Neck&Back or Back&Hip
- C: Adjustable leg band

### HOW TO PUT ON



1. Place the brace on the dog's leg. Attach the leg band first to keep it in place. Adjust the band to best fit
2. Connect the knee brace to the 2-in-1 Massage Brace used for Neck&Back or Back&Hip like the image.

3